

MyPurMist Quickstart Guide

1. Attach mask (insert at a 45° angle (see manual)).
2. Fill water reservoir to near the top with **MyPurFill** water or distilled water. Place cap over reservoir. Let water soak in for 30 seconds. Each full reservoir provides approx 25 minutes of steam therapy.
3. Plug the unit in and Push the “ON” button: Enjoy the benefits of **MyPurMist!**
4. Adjust the temperature using the UP and DOWN button. There are four blue lights for temperature and one red light for “out of water” indication.
5. When done, push the power button for “OFF”. The blue lights will turn off and the fan will run for 10 minutes to dry the internal parts (do not unplug). No cleaning required.
6. If a RED “water out” light comes on, refill the water reservoir to near the top.
7. **Always refill the water reservoir to near the top before each use of MyPurMist.**



For technical support or warranty claims please email care@mypurmist.com (preferred) or call 1-866-925-6710. Do not return it to the store or online merchant. For non-technical issues call 1-855-697-8764.



Secure your warranty today by registering your MyPurMist inhaler. Go to www.MyPurMist.com/register

